Manual LCC RESPIRA 1

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Manual LCC RESPIRA 1

English version

*Low Cost & Comfortable Powered
Air Purifying Respirator*

Copyright 2020

Manual LCC RESPIRA 1

LCC Respira 1 is a Low Cost & Comfortable Powered Air Purifying Respirator, designed as respiratory Personal Protective Equipment (PPE) for medical staff dealing with *highly infectious disease*. As an air-purifying respirator (APR) it can also be used in other respiratory-hazardous condition.

The equipment meets the requirements and testing by BPFK, Indonesian Ministry of Health.

A. Components and Functions

The equipment is made of a mask mask, connected by inhalation hose (CPAP), to the filter and electronic section.

LCC-Respira is equipped with HEPA filter (can be replaces with other filter in accordance to the use requirement), able to sieve and filter micro particles and virus, without stifling or suffocating the user. The filter is replacable after certain length of usage, ensuring safety in long term usage. Clean air is pumped to the mask to ease breathing,

The electronic section employs an electromotor to pump the filtered air. Electric controller is embedded to regulate the blower rate, according to adjustable settings. On intermediate flow, battery powering the motor can last up to 8 hours.

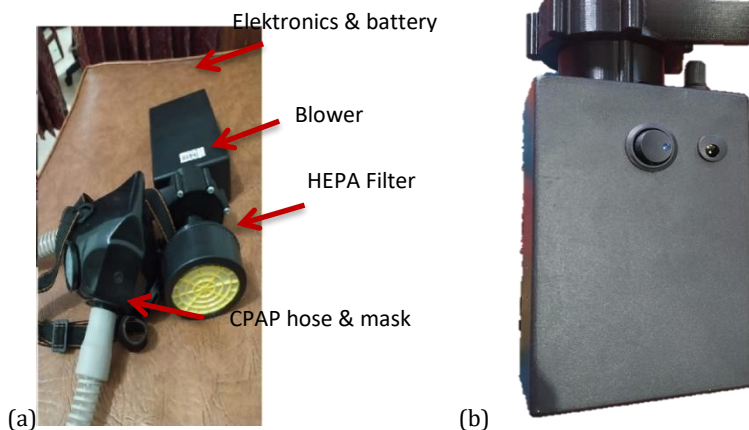
Figure 1. (a) Components, (b) Electronics & battery container

Manual LCC RESPIRA 1

B. Instalation

1. Open the HEPA Filter closure by rotating the lid. Replace filter after 40 hours of usage (Figure 2.a)
2. Place the new filter (Figure 2.b), and close the lid tightly (Figure 2.c).
3. Attach the CACP hose to mask (Figure 2.d), and to motor container (Figure 2.c) so as to connect all components (Figure 3.a).
4. Insert motor container (Figure 2.c) to the waist bag.
5. LCC Respira 1 is ready to use (Figure 3.b).

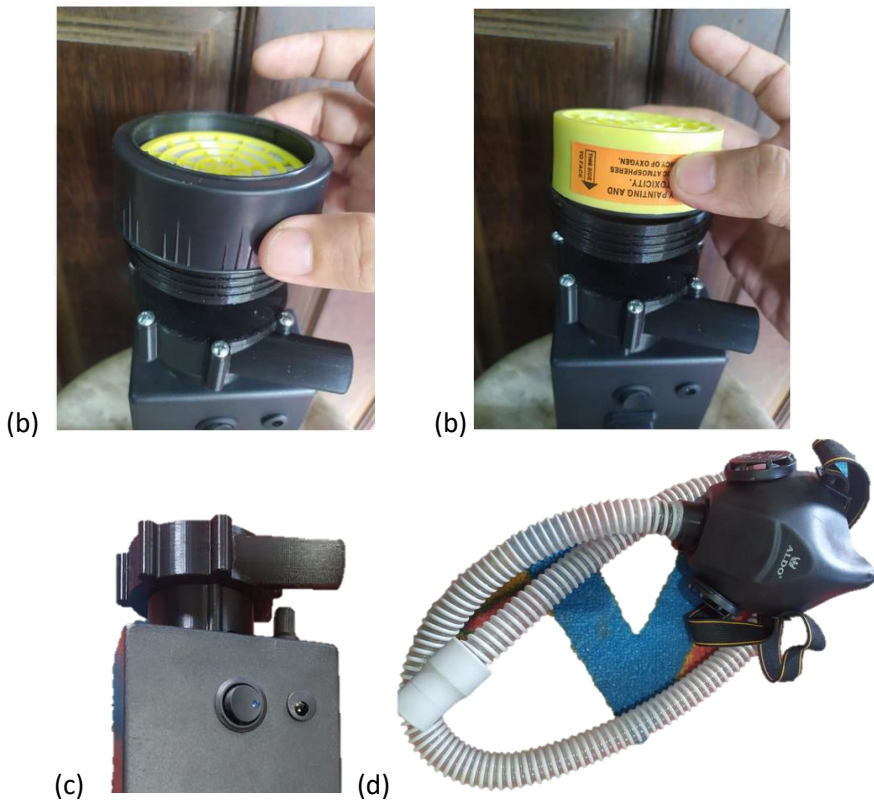
Figure 2. (a) Opening filter closure, (b) Replacing filter, (c) Ready to blower and motor, (d) CPAP dan Masker.

Manual LCC RESPIRA 1

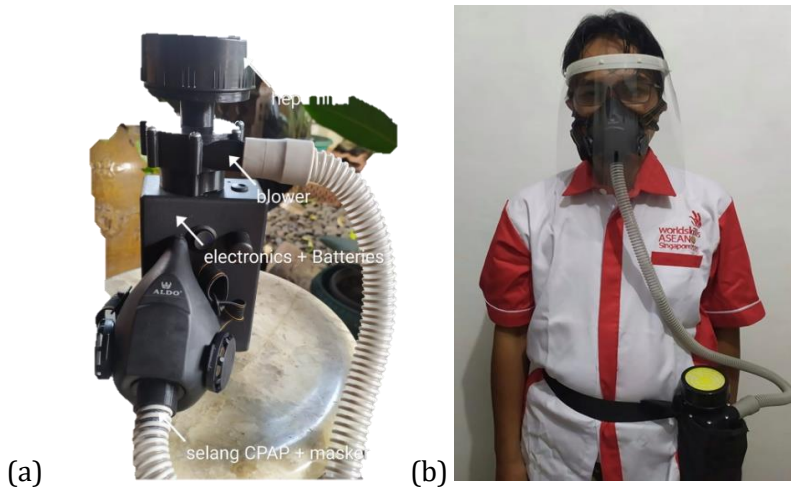
Figure 3. (a) complete LCC Respira 1, (b) LCC Respira 1 ready to use

C. Operational

1. Make sure the Hepa Filter is well installed and still good for use.
2. Make sure the battery is full charged. The usage capacity is 8 hours on intermediate blowing level. Recharge the battery after each use.
3. Make sure all the components are perfectly connected.
4. Turn the machine on using the on/off button on elektronik container (Figure 4)

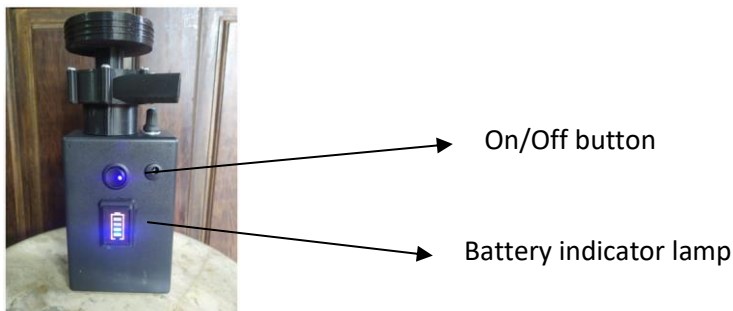
Figure 4. On/off button and indicator lamp.

5. As the motor is on, air is filtered and blown.
6. The filtered air is blown through CPAP hose to the mask.
7. Check the air condition within the mask whether it is stable
8. As the clean air stabilized, LCC Respira is in perfect working function.

Manual LCC RESPIRA 1

D. Battery replacement

Battery replacement might be necessary after periods of use, when recharging is unsuccessful. This might due to impairment on the lithium battery parts.

To replace the battery:

1. Open the electronic container
2. Take out the batteries
3. Replace with a new set of batteries
4. Close the container tightly
5. Charge the battery until indicator lamp shown that the battery is full charged
6. The equipment is ready to reuse.

Figure 5. Battery replacement

D. Procedures to use LCC Respira 1

1. Make sure all components are complete and rightly installed.
2. Put on the waist bag of electronic container, on right and comfortable position (Figure 6).
3. Connect all parts.
4. Put on the half mask to cover nose and mouth by tightening the *head loop*.
5. Put on the face protection. Headloop is adjustable to head measurement.
6. Breath normally.

Manual LCC RESPIRA 1

Figure 6. Putting on LCC-Respira 1 (front and side look)

Table 1. Specification of LCC Respira 1

| Components | Specifications |
|-----------------------|--|
| Air flow | Adjustable 7-15 LPM |
| Bacterial filter | HEPA |
| Blower material | anti-bacterial PLA |
| Mask & hose weight | 150 gr |
| Electronic box weight | 480 gr |
| Battery | - Rechargeable 3S Lithium Ion Battery - Indikator & alarm baterai |
| Battery weight | 320 gr |
| Charging time | Kurang dari 3 jam |
| Operation time | - low air flow 8-9 jam - high airflow 5-7 jam |